

VIP treatment for high success rate embryos in IVF



Multiroom Incubator for IVF

Get superior stability with the MIRI® Multiroom Incubator.

Make the most of its integrated tri-gas system and six independent chambers.



table of contents

MIRI® Multiroom Incubator Parts and General Features
Quality Control Features
Accessories
General Specifications
Ordering Information





"An advanced temperature regulation system for routine/long-term embryo incubation at your fingertips"

The MIRI® has six (6) chambers which are completely independent of each other. This is ideal because any disruption (e.g. temperature drop after opening the lid) has zero impact on the rest of the system. Furthermore, calibration is so much simpler because there is no crossover of heat from adjacent chambers.



Hand-in-hand with the best in incubation.

This equipment is a CE-marked device and is in conformity with the essential requirements of the Medical Devices Directive 93/42/EEC as amended by Directive 2007/47/EC.

FEATURES:

Heated Lid

- Prevents condensation
- Enhances temperature regulation and recovery
- Excellent uniformity between the top and the bottom
 - Temperature accuracy: ± 0.2 °C
 - Temperature uniformity: ± 0.2 °C

Heated bottom

- Provides direct heat transfer to the cultures through the optimization plate for stable heat regulation.
- Removable Heating Optimization Plate with wide selection of inserts.
 - Temperature accuracy: ± 0.2 °C
 - Temperature uniformity: ± 0.2 °C

Optional SAFE Sens Integration

For continuous pH monitoring. See page 9 for more info.

Six (6) Chambers

Completely individual chambers for easier calibration, faster recovery, less disruption, and prevents cross-contamination.

Control Panel Buttons and LED Display

Has large LED display that can be easily seen from a distance. The simple 4-button control panel allows for easy and intuitive operation.

Mute Button

Temporarily mutes alarm messages and sound for five (5) minutes

Maximize embryo growth potential by providing VIP treatment



Oxygen range: 5-20% Carbon dioxide range: 1.9-10% Temperature range: 25 Ambient to 40°C Gas recovery: less than 3 minutes Temperature recovery: less than 1 minute

Common Stressors:

- Temperature fluctuations
- Gas concentration fluctuations
- Non-optimal pH
- Volatile Organic Compounds (VOCs)

Elevated O, concentration isn't always a good thing

While oxygen (O_2) is necessary for normal aerobic metabolism, it is a double-edged sword as it can harm the developing embryo through oxidative damage. Recent studies highlight the benefit of having suppressed oxygen levels when incubating human embryos reflecting the natural low oxygen conditions in the womb.

Shhh... Do not disturb

The MIRI® has an overall design that provides cultured embryos a minimum-stress environment. The independent chamber system prevents cross-contamination while HEPAVOC filtration cleans the airstream. The small chamber volumes and direct heat regulation further translate to faster temperature and gas recovery.

Fast Recovery




There are many advantages to using benchtop multiroom incubators. One important benefit is the speed of recovering temperature and gas parameters after opening a chamber.

The little details count



IVF practitioners deal with precious, fragile and sensitive embryos, and often, the little details make a big difference. The MIRI® has a large LED display that can be easily seen from a distance. Also, the glass lid tops, can be written on — a very useful feature for organization.

Provide total control of the gas phase environment

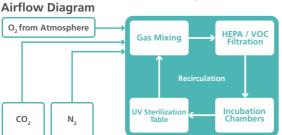

The built-in gas mixer and the high-performance CO_2 and O_2 sensors allow accurate control of gas phase composition within the chambers.

The MIRI® is built with Excellent Quality Control Features

The MIRI® has reliable gas mixing system that allows gas phase flexibility.



Premixed gas is NOT required

 $MIRI^{10}$ can accept pure gas (and even premixed gas if desired). The use of 100% CO_2 and 100% N_2 allows the incubator to achieve steady-state gas condition faster than premixed gases. This is important as this will stabilize the pH during embryo development.

High quality airstream via HEPA/VOC filter +

The filter module can be easily replaced once used.

The gas in the MIRI® is continuously recirculated through a HEPAV VOC filter. A UV-C light (254 nm) sterilizes the airstream before passing through the filter.

A suite of IVF-essential features

There is a BNC connection for pH measurement, USB communication port, and port for external alarm monitoring.

The MIRI® can be connected to a PC to avail of data logging via the supplied software included. Connections to external alarm monitoring systems and pH measurements are also possible.

Stress-free validation of chamber parameters



PT1000 temperature sensors are built-in, which are completely independent from the main circuitry. Gas sampling ports are likewise available for all 6 chambers.

The MIRI® can be connected to an external device such as the Esco MIRI® GA for gas and temperature validation.





Full-featured and user-friendly

Control panel, display, and data logging software



Complete parameters are displayed, Histories of any alarm events are logged.



The data logger stores continuous performance data of the machine throughout its use. These can be viewed in graphs



Conditions that put the MIRI® into alarm state are recorded It is possible for the software to send email alerts as well.

The MIRI® can be connected to an easy-to-use, feature-packed data logging software installed on any PC and connected via USB.

Multiple machines can be connected and managed from a single computer. All real-time parameters of the machine can be conveniently viewed. These include the temperature of all monitored temperature and gas concentration points, gas input pressures, gas flow rates, current gas readings, and all set points.

All performance data of the machine including alarms are continuously logged and can be viewed in graphs. The data logger also automatically generate reports weekly which makes it more convenient for the user.

Accessories



For MIRI® with integrated SAFE Sens (optional), order a different plate with hole to accomodate the SAFE Sens sensor (see ordering information on page 11).

The dishes fit into the inserts so that the heat is directly transferred to the media.

Heating optimization plates

Each chamber contains a heating optimization plate to facilitate heat transfer directly to the culture

- Has inserts to fit various dish sizes
- Removable for easy cleaning





LifeGlobal® GPS Dishes





NiproTM





SparMED Oosafe

Total Capacity

Heating plates customized for several types of dishes:

- 4 x Falcon® Ø 50/60 mm
- 8 x Falcon® Ø 35 mm
- 4 x Nunc™ Ø 54/60 mm
- 8 x Nunc™ Ø 35 mm
- 4 x Vitrolife Dishes
- 4 x LifeGlobal® GPS Dishes
- 4 x Nipro™
- 4 x SparMED Oosafe®

*Applicable for MIRI® only

SAFE Sens* Continuous pH Monitoring

The MIRI® can be installed with an integrated SAFE Sens technology for fast, effective, and non-invasive continuous pH monitoring product for *in vitro* fertilization (IVF) procedures.

The SAFE Sens technology employs an optical fluorescent measurement technology, used in combination with disposable sensors, which accurately and reliably monitors the pH of small volumes of fluids such as the media used in IVF.



Key Features



Continuous pH measurement

- Reading and recording every 30 minutes (default setting adjustable).
- Single use sensor probe for up to seven (7) days of pH readings.



Data Logging System**

- Data Logging and user alarms.
- Each TrakStation® can be connected to multiple incubators.



Compact and Efficient

- No more unnecessary openings of your incubator for spot pH measurement.
- Only requires 100 μL of media + 50 μL of oil.

[•] Intel Core 2 Duo or AMD Athlon X2 at 2.4 GHz processor • 4Gb RAM • 15Gb Hard Disk space • Integrated Video Card • Monitor with resolution 1024 x 768 • Windows 7 pro/ 8 Pro/ 10 OS with 64 Bit architecture • USB 3.0 port for each connected device



MRA-1014 - Stacking frame for 2 units



MIRI® Stacking Frame with drawer

MIRI® Stacking Frame

MIRI® has a stacking system to maximize space in your IVF laboratory.

^{*} SAFE Sens is a trademark brand of Blood Cell Storage, Inc. (BCSI). SAFE Sens integration is currently offered as a factory-installed option.
**Minimum system requirements for Data Logger PC/Tablet:

General Specifications



MIRI® Multiroom Incubator

Product Code	MRI-6A10	
Overall Dimensions (W x D x H)	700 x 580 x 160 mm (27.6" x 22.9" x 6.3")	
Power Supply	115 / 230V, 50/60 Hz	
Power Consumption	280 W	
Temperature Control Range	25 - 40°C	
*Gas Consumption (CO ₂)	<2 L/h	
**Gas Consumption (N ₂)	<12 L/h	
CO ₂ Control Range	1.9 - 10%	
O ₂ Control Range	5 - 20%	
Input Gas Pressure (CO ₂)	0.6 bar (8.7 psi)	
Input Gas Pressure (N ₂)	0.6 bar (8.7 psi)	
Net Weight	35 kg (77.2 lbs)	
Shipping Weight	40 kg (88.2 lbs)	
Shipping Dimension	840 x 735 x 300 mm (33.1" X 29" x 11.9")	

MIRI® Stacking Frame Dimensions				
Stacking frame for 2 units (W x D x H)	717 x 572 x 632 mm (28.2" x 22.5" x 24.9")			
MIRI® Stacking Frame with Drawer (W x D x H)	715 x 650 x 363 mm (28.1" x 25.6" x 14.3") On full opening of the drawer: 715 x 1245 x 363 mm (28.1" x 49" x 14.3")			

- * Under normal condition (CO $_2$ set point reached at 5.0%, all lids closed) ** Under normal condition (O $_2$ set point reached at 5.0%, all lids closed)



Ordering Information

ITEM CODE	MODEL CODE	DESCRIPTION
Unit		
2070047	MRI-6A10-8	MIRI® Incubator, 230V, 50/60Hz
2070048	MRI-6A10-9	MIRI® Incubator, 115V, 50/60Hz
2070086	MRI-6A10-SS-8	MIRI® Incubator, with SAFE Sens for pH measurement, 230V, 50/60Hz
2070087	MRI-6A10-SS-9	MIRI® Incubator, with SAFE Sens for pH measurement, 115V, 50/60Hz
1320045	MRI-GA	$\rm MIRI^{\otimes}~GA~CO_{_2}/~O_{_2}~\&$ Temperature Validation Unit, 115V / 230V
Accessories		
1320191	TBA	SAFE Sens TrakStation, a tablet with SAFE Sens Software, for pH monitoring
1320011	MRA-1007	HEPAVOC filter (recommended to be replaced every 3 months)
1320018	MRA-1014	Stacking frame for 2 units
1320226	TBA	Stacking Frame for 2 Units, With Drawer at the Bottom
1081277	TBA	SAFE Sens SV2 Sensor, Pack of 10 pieces (shelf-life 12 months)
1081278	TBA	SAFE Sens QC2 Alignment Tool

Accessories

ITEM CODE	MODEL CODE	DESCRIPTION
1320003	TBA	Insert for Falcon® Dishes
1320004	TBA	Insert for Nunc™ Dishes
1320070	TBA	Insert for Vitrolife Dishes
1320099	TBA	Insert for Nipro™ Dishes
1320100	TBA	Insert for LifeGlobal® GPS Dishes
1320101	TBA	Insert Without Footprint for Plain Dishes
1320118	TBA	Insert for SparMED Oosafe®
1320219	TBA	Insert for Falcon® Dishes, with hole for SAFE Sens
1320220	TBA	Insert for $Nunc^{TM}$ Dishes, with hole for SAFE Sens
1320221	TBA	Insert for Vitrolife Dishes, with hole for SAFE Sens
1320222	TBA	Insert for $Nipro^TM$ Dishes, with hole for SAFE Sens
1320223	TBA	Insert for LifeGlobal® GPS Dishes, with hole for SAFE Sens
1320224	TBA	Insert Without Footprint for Plain Dishes, with hole for SAFE Sens
1320225	TBA	Insert for SparMED Oosafe®, with hole for SAFE Sens

ESCO GLOBAL NETWORK





Esco Medical Products:

MIRI® TL6 Time-Lapse Incubator MIRI® TL12 Time-Lapse Incubator

MIRI® Multiroom Incubator

MIRI® II Multiroom Incubator

Mini MIRI® Incubator

Esco Multi-Zone ART Workstation

Esco Multi-Zone ART Workstation Class II

CelCulture® CO2 Incubator


MIRI® GA (Gas and Temperature Validation Unit) Anti-Vibration Table (AVT)

Versati[™] Tabletop Refrigerated Centrifuge CultureCoin®

Infertility is a problem that has a significant social, psychological, and economic impact on afflicted individuals and couples. It is a global concern that knows no race or creed. It has been estimated that 1 in 6 couples struggle with infertility at least once in their

Esco Medical is one of the divisions of the Esco Lifesciences Group. We provide innovative technological solutions for fertility clinics and laboratories. We aim to become the leading manufacturer of high-quality equipment such as long-term embryo incubators, ART workstations, anti-vibration tables, and time-lapse incubators.

Our products are designed with the Silent Embryo Hypothesis as a guiding principle. The Silent Embryo Hypothesis states that the less disturbed an embryo can remain the better its developmental potential will be. Most of our products are designed in Denmark and made in the EU. Our primary focus is to increase pregnancy success rates and patient satisfaction.

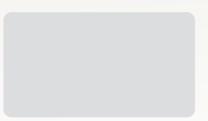



21 Changi South Street 1 • Singapore 486 777 Tel +65 6542 0833 • Fax +65 6542 6920 medical@escoglobal.com • www.esco-medical.com

Esco Global Offices: Bangladesh | China | Denmark | Germany | Hong Kong | Indonesia | Italy | Lithuania | Malaysia | Myanmar | Philippines | Russia | Singapore | South Africa | South Korea | Taiwan | Thailand | UAE | UK | USA | Vietnam





















