

Excellent Embryo Hosting for High Success Rates

ESCO
MEDICAL

MIRI® Laminar Flow Cabinet



MIRI[®] Laminar Flow Cabinet

Comes with advanced features that will help alleviate the workflow in your laboratory.



Table of Contents

- MIRI® Laminar Flow Cabinet4
- Workstation Special Features6
- Engineering Drawings8
- General Specifications12
- Ordering Information14
- Accessories16



MIRI[®] Laminar Flow Cabinet



Esco MIRI[®] Laminar Flow Cabinet brings your laboratory a sophisticated approach to handle your sample.

The MIRI® Laminar Flow Cabinet has been designed with the capability to maintain a controlled work surface for laboratory applications requiring a clean and sterile workspace, which is achieved through the use of vertical laminar airflow pattern.

Clean filtered air travels from the top of the cabinet downwards onto the working surface. Laminar flow cabinets are ideal to be used for our customers who requires a clean and sterile workspace to provide good protection towards the sample.

FEATURES:

ISO Class 3 Cleanliness

Esco cabinets deliver ISO Class 3 air cleanliness within the working zone, which is much cleaner than the competition's Class 4 designation. This is achieved through the employment of 100% polyester fiber pre-filter with 85% arrestance and the main HEPA/ULPA filtration system.

| Air Cleanliness Standards (ISO 14644.1, Air Cleanliness Particle Limits) (No. of Particles / m ³) | | | | | | |
|---|-------------------|---------|---------|---------|---------|---------|
| Particle Size (µm) | Cleanliness Class | | | | | |
| | Class 1 | Class 2 | Class 3 | Class 4 | Class 5 | Class 6 |
| 0.1 | 10 | 100 | 1000 | 10000 | 100000 | 1000000 |
| 0.2 | 2 | 24 | 237 | 2370 | 23700 | 237000 |
| 0.3 | 1 | 10 | 102 | 1020 | 10200 | 102000 |
| 0.5 | - | 4 | 35 | 352 | 3520 | 35200 |
| 1.0 | - | - | 8 | 83 | 832 | 8320 |
| 5.0 | - | - | - | 3 | 29 | 293 |

Built-in Mini-PC + Monitor (Optional)


- User may opt to have a touchscreen monitor to be integrated in the cabinet.
- Allows the user to connect with the microscope through a camera.
- Provides the user easier analysis of the specimen being observed under the microscope.
- Provides the possibility to integrate MIRI® Evidence RFID Witnessing and Traceability System.

Integrated Microscope Pole (Optional)

- Enables the user to install the microscope easier.
- Compatible with the most commonly used microscope brand.

Anti-Microbial Growth Mechanism

Most of Esco manufactured products are coated with a silver-ion-impregnated ISOCIDE™ powder, can effectively inhibit and eliminate unwanted microbial growth. Therefore it is able to provide further sample protection against possible contamination.

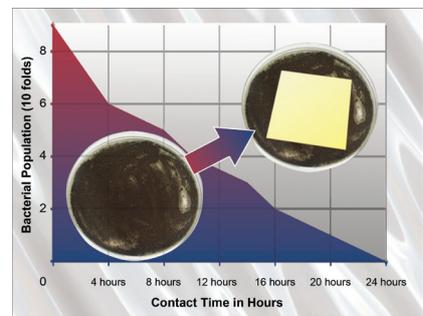



Advanced Cabinet designed to help the workflow in your laboratory setting

Integrated Light Source (Optional)


The Cabinet comes with its own LED light source which provides the user with better energy efficiency.

Outstanding Sample Protection

The MIRI® Laminar Flow operates by applying the vertical laminar airflow pattern, in which the clean filtered air will travel downward from the filter located on top of the cabinet to the working surface. The HEPA/ULPA filtration system provides a particulates filtering capability of 99.9995%, maximising the air quality that goes onto the working zone.

Energy Efficient

MIRI® Laminar Flow Cabinet comes with the latest generation of ECM blower which prevails in the minimum cabinet power consumption without sacrificing the cabinet's performance.

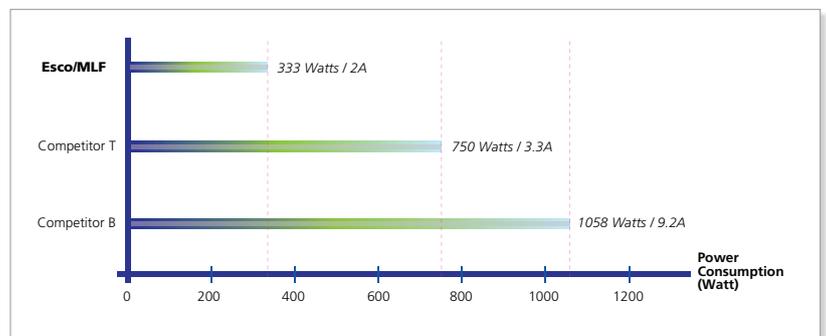

Sentinel Gold Microprocessor Control System



Displays all safety information on one screen; there's also the standby mode which allows users to switch to an energy saving mode when the cabinet is not in use.



Quiet Operation

MIRI® Laminar Flow has a state-of-the-art design and features resulting in very low noise and vibration level that makes the workstation suitable for sensitive microscopic work. The high-efficiency ECM blower creates less noise and causes minimal vibrations on the cabinet, thus providing better working conditions for the user.

- Reduce the possibility of distraction due to loud noise, improving work concentration.

Energy Efficient

Electrical wires can be easily arranged without disturbing the working area layout of the user during sample handling since the MIRI® Laminar Flow cabinet comes with cable pass-through channel which enables the wires to run through the back wall.



Cable Pass-Through Tunnel

Cable Wire

MIRI® Laminar Flow Cabinet




- Outstanding sample protection.
- Low noise and energy-efficient ECM blowers.
- Provision for microscope.*
- Integrated transmitted LED light source.*
- ISOCIDE™ Anti-microbial growth mechanism.
- Built in Mini-PC + Monitor*
- Wide range of cabinet size selection, between 3 feet, 4 feet, 5 feet or 6 feet

*Microscope, LED light source and the built-in monitor are optional accessories. Be sure to specify the configuration you desire when ordering.




MIRI[®] Laminar Flow Workstation

MLF-3D-MON w/LS

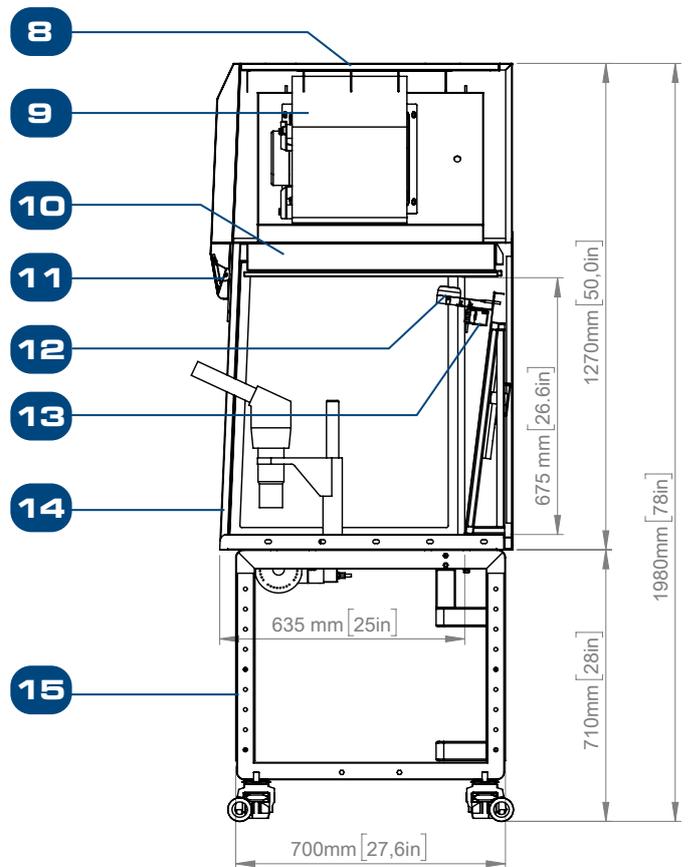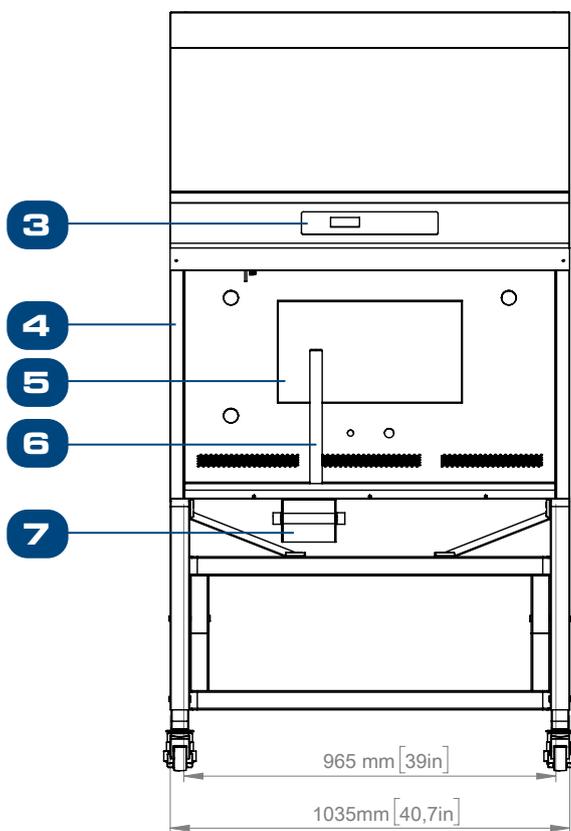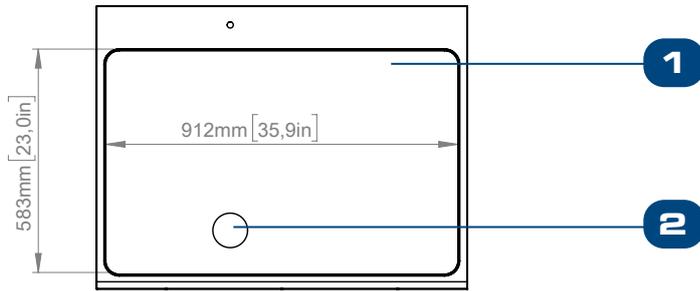

- | | |
|---|--|
| <ol style="list-style-type: none"> 1. Tabletop 2. Transmitted Light Source 3. Esco Sentinel™ Gold 4. Tempered Glass Side Panel 5. Monitor (Optional) 6. Microscope Mounting Pole (Microscope not Included) 7. Light Source | <ol style="list-style-type: none"> 8. Pre-Filter 9. ECM Blower 10. HEPA/ULPA Filter 11. LED Light 12. Airflow Sensor 13. UV Light (Optional) 14. Polycarbonate Cover (Optional) 15. STA-3A0 (TELESCOPIC) |
|---|--|


MLF-4D-MON w/LS

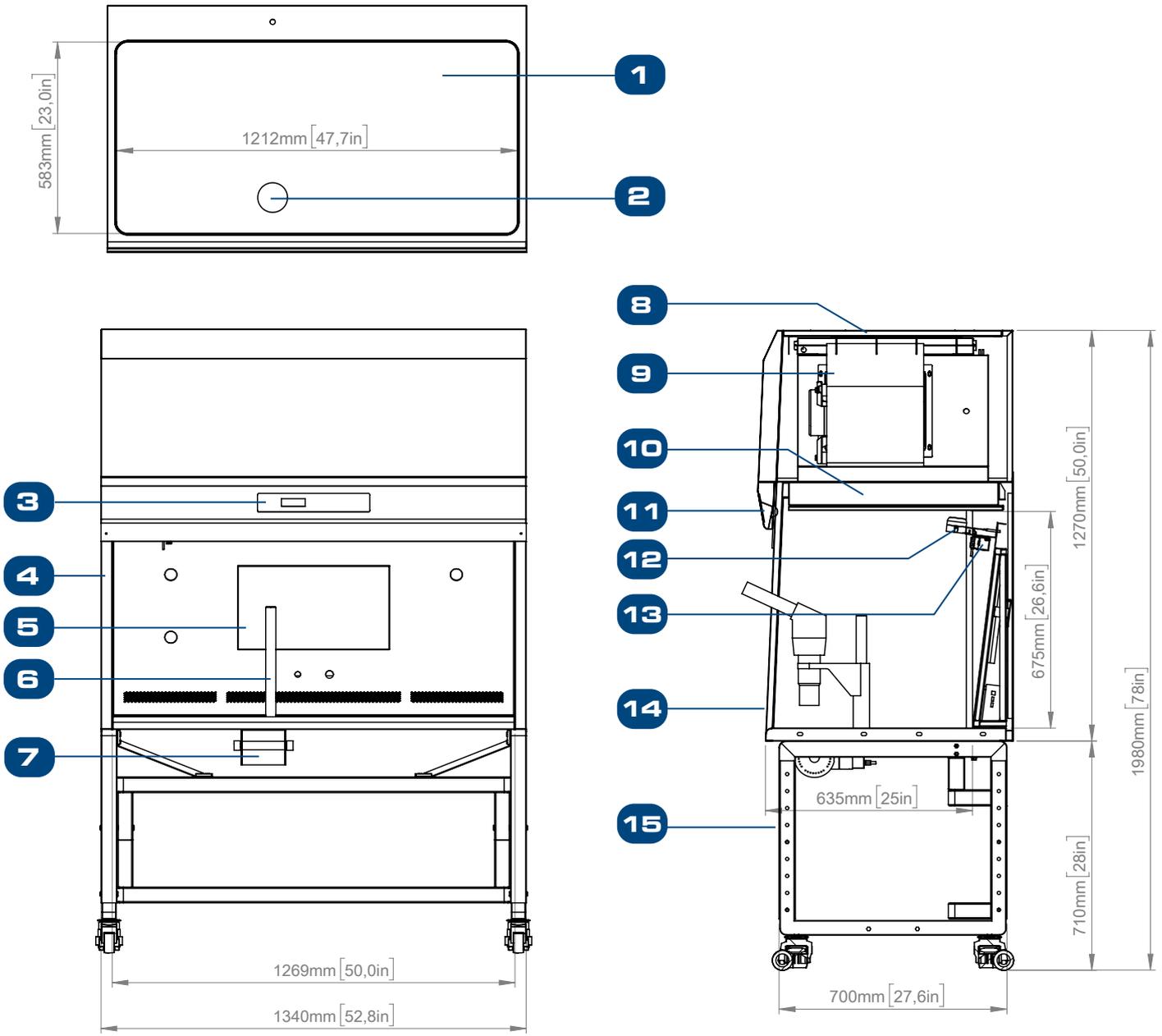

- | | |
|---|--|
| <ul style="list-style-type: none"> 1. Tabletop 2. Transmitted Light Source 3. Esco Sentinel™ Gold 4. Tempered Glass Side Panel 5. Monitor (Optional) 6. Microscope Mounting Pole (Microscope not Included) 7. Light Source | <ul style="list-style-type: none"> 8. Pre-Filter 9. ECM Blower 10. HEPA/ULPA Filter 11. LED Light 12. Airflow Sensor 13. UV Light (Optional) 14. Polycarbonate Cover (Optional) 15. STA-4A0 (TELESCOPIC) |
|---|--|




MIRI[®] Laminar Flow Workstation

MLF-5D-MON w/LS

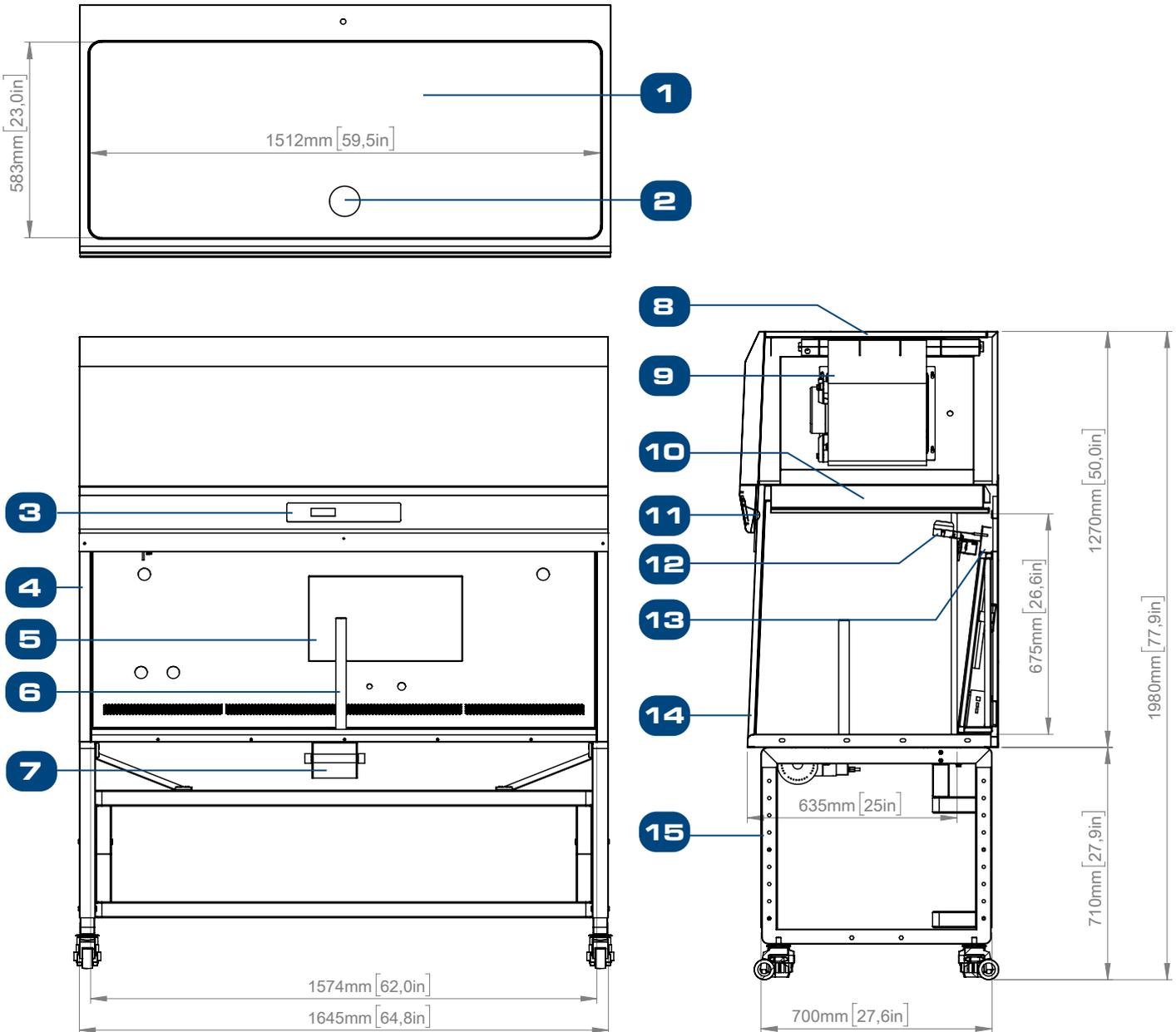

- | | |
|--|------------------------------------|
| 1. Tabletop | 8. Pre-Filter |
| 2. Transmitted Light Source | 9. ECM Blower |
| 3. Esco Sentinel™ Gold | 10. HEPA/ULPA Filter |
| 4. Tempered Glass Side Panel | 11. LED Light |
| 5. Monitor (Optional) | 12. Airflow Sensor |
| 6. Microscope Mounting Pole (Microscope not Included) | 13. UV Light (Optional) |
| 7. Light Source | 14. Polycarbonate Cover (Optional) |
| | 15. STA-5A0 (TELESCOPIC) |

MLF-6D-MON w/LS

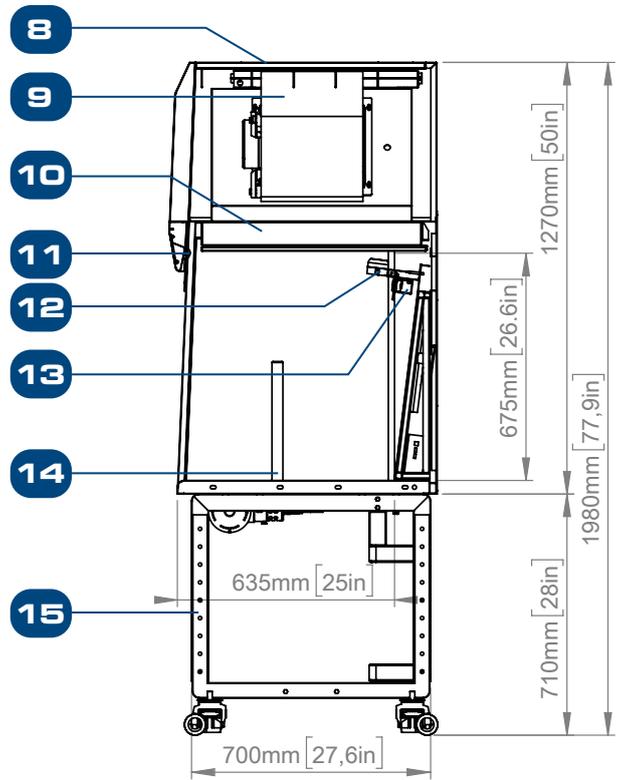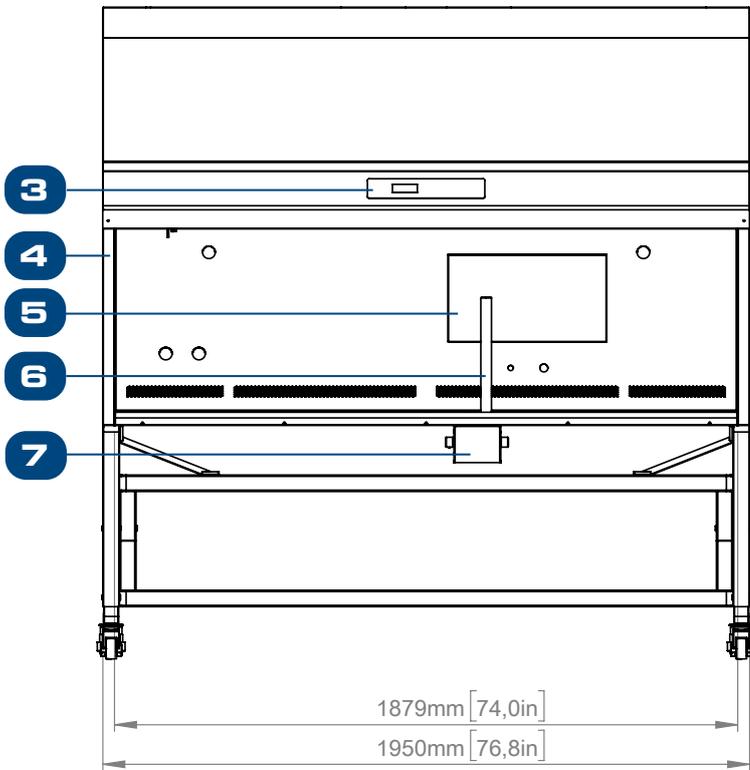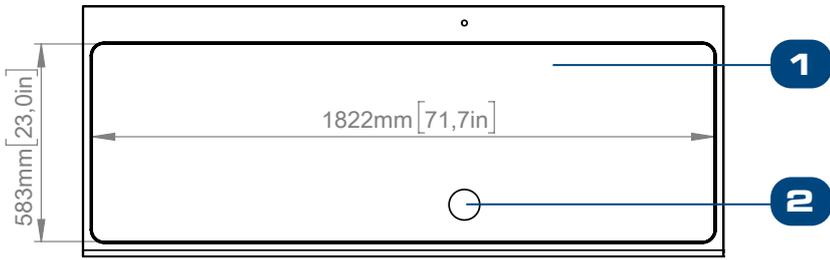

- 1. Tabletop
- 2. Transmitted Light Source
- 3. Esco Sentinel™ Gold
- 4. Tempered Glass Side Panel
- 5. Monitor (Optional)
- 6. Microscope Mounting Pole (Microscope not Included)
- 7. Light Source
- 8. Pre-Filter
- 9. ECM Blower
- 10. HEPA/ULPA Filter
- 11. LED Light
- 12. Airflow Sensor
- 13. UV Light (Optional)
- 14. Polycarbonate Cover (Optional)
- 15. STA-6A0 (TELESCOPIC)

General Specifications

MIRI® Laminar Flow Cabinet

| Model | | MLF-3D_ | MLF-4D_ | MLF-5D_ | MLF-6D_ |
|---|-------------------|---|---|---|---|
| Work Area Dimension (W x D x H) | | 965 x 635 x 675 mm (39.0" x 25.0" x 26.6") | 1269 x 635 x 675 mm (50.0" x 25.0" x 26.6") | 1574 x 635 x 675 mm (62.0" x 25.0" x 26.6") | 1879 x 635 x 675 mm (74.0" x 25.0" x 26.6") |
| External Dimensions without Support Stand (W x D x H) | | 1035 x 760 x 1270 mm (40.7" x 29.9" x 50.0") | 1340 x 760 x 1270 mm (52.8" x 29.9" x 50.0") | 1645 x 760 x 1270 mm (64.8" x 29.9" x 50.0") | 1950 x 760 x 1270 mm (76.8" x 29.9" x 50.0") |
| External Dimension with "STL" Type Support Stand (W x D x H) | | 1035 x 760 x 1980 mm (40.7" x 29.9" x 78.0") | 1340 x 760 x 1980 mm (52.8" x 29.9" x 78.0") | 1645 x 760 x 1980 mm (64.8" x 29.9" x 78.0") | 1950 x 760 x 1980 mm (76.8" x 29.9" x 78.0") |
| Cabinet Construction | Main Body | 1.2 mm (0.05") 18-gauge electro-galvanized steel with white oven baked epoxy-polyester powder coated finish | | | |
| | Work Zone | 1.2 mm (0.05") 18-gauge stainless steel, grade 304, with 4B finish | | | |
| | Side Walls | UV-absorbing tempered glass 5mm (0.2"), colourless and transparent | | | |
| | Sash | | | | |
| Power Supply | MLF-_D8 | 220-240V, 50/60Hz | | | |
| | MLF-_D9 | 110-130V, 50/60Hz | | | |
| Motor Type | | ECM Motor | | | |
| Pre-Filter | | Disposable and non-washable 100% polyester fiber with 85% arrestance, EU-3 rated | | | |
| Filter Efficiency | | HEPA/ULPA filtration with 99.9995% efficiency | | | |
| ISO Classification | | ISO Class 3 | | | |
| Noise Level Reading* | | ≤48 dB(A)* | | | |
| Control System | | Esco Sentinel™ Gold Microprocessor Controller | | | |
| Microscope Pole Provision | | Position for 1 microscope | | | |
| Transmitted Light Source | | 1 set | | | |
| Advanced Option | | Touchscreen Monitor** | | | |
| Net Weight | | ±191 kg | ±206 kg | ±221 kg | ±236 kg |
| Cabinet Weigh with Support Stand | | ±265 kg | ±280 kg | ±295 kg | ±310 kg |

*Noise reading in open field condition/ anechoic chamber. Noise reading in normal room varies by room size, layout, and background noise, but may reach roughly 3-4 dBA above these values.

** Monitoring screen system comes as accessory, you are required to specify in your quotation

Ordering Information

| Item Code | Model Code | Description |
|-----------------------------|-------------|---|
| Cabinet without Lightsource | | |
| 2070255 | MLF-3D8 | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 220-240V, 50/60Hz |
| 2070259 | MLF-3D9 | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 110-130V, 50-60Hz |
| 2070256 | MLF-4D8 | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 220-240V, 50/60Hz |
| 2070260 | MLF-4D9 | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 110-130V, 50-60Hz |
| 2070257 | MLF-5D8 | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 220-240V, 50/60Hz |
| 2070261 | MLF-5D9 | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 110-130V, 50-60Hz |
| 2070258 | MLF-6D8 | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 220-240V, 50/60Hz |
| 2070262 | MLF-6D9 | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 110-130V, 50-60Hz |
| 2070247 | MLF-3D8-MON | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070251 | MLF-3D9-MON | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 110-130V, 50-60Hz with screen monitor integration |
| 2070248 | MLF-4D8-MON | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070252 | MLF-4D9-MON | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 110-130V, 50-60Hz with screen monitor integration |
| 2070249 | MLF-5D8-MON | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070253 | MLF-5D9-MON | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 110-130V, 50-60Hz with screen monitor integration |
| 2070250 | MLF-6D8-MON | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070254 | MLF-6D9-MON | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 110-130V, 50-60Hz with screen monitor integration |

Ordering Information

| Item Code | Model Code | Description |
|--------------------------|------------------|---|
| Cabinet with Lightsource | | |
| 2070217 | MLF-3D8 w/LS | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 220-240V, 50/60Hz |
| 2070220 | MLF-3D9 w/LS | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 110-130V, 50-60Hz |
| 2070218 | MLF-4D8 w/LS | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 220-240V, 50/60Hz |
| 2070221 | MLF-4D9 w/LS | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 110-130V, 50-60Hz |
| 2070182 | MLF-5D8 w/LS | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 220-240V, 50/60Hz |
| 2070222 | MLF-5D9 w/LS | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 110-130V, 50-60Hz |
| 2070219 | MLF-6D8 w/LS | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 220-240V, 50/60Hz |
| 2070236 | MLF-6D9 w/LS | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 110-130V, 50-60Hz |
| 2070238 | MLF-3D8-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070242 | MLF-3D9-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 3ft (0.9 meter), 110-130V, 50-60Hz with screen monitor integration |
| 2070239 | MLF-4D8-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070243 | MLF-4D9-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 4ft (1.2 meter), 110-130V, 50-60Hz with screen monitor integration |
| 2070240 | MLF-5D8-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070244 | MLF-5D9-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 5ft (1.5 meter), 110-130V, 50-60Hz with screen monitor integration |
| 2070241 | MLF-6D8-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 220-240V, 50/60Hz with screen monitor integration |
| 2070245 | MLF-6D9-MON w/LS | Esco MIRI® Laminar Flow Cabinet, 6ft (1.8 meter), 110-130V, 50-60Hz with screen monitor integration |

Accessories

| Item Code | Model Code | Description |
|-----------|---------------------|---|
| 5131566 | STA-3B0 | Support Stand w/ Caster Wheels and Leveling Feet (28" and 36") for 3ft cabinets |
| 5131395 | STA-4B0 | Support Stand w/ Caster Wheels and Leveling Feet (28" and 36") for 4ft cabinets |
| 5131567 | STA-5B0 | Support Stand w/ Caster Wheels and Leveling Feet (28" and 36") for 5ft cabinets |
| 5131568 | STA-6B0 | Support Stand w/ Caster Wheels and Leveling Feet (28" and 36") for 6ft cabinets |
| 5170251 | UV-15A-L | UV lamp for 3ft MLF cabinet |
| 5170255 | UV-30A-L | UV lamp for 4ft, 5ft, and 6ft MLF cabinet |
| 5171182 | FC-MLF-3A | Polycarbonate front cover for MLF cabinet 3ft |
| 5171183 | FC-MLF-4A | Polycarbonate front cover for MLF cabinet 4ft |
| 5171245 | FC-MLF-5A | Polycarbonate front cover for MLF cabinet 5ft |
| 5171184 | FC-MLF-6A | Polycarbonate front cover for MLF cabinet 6ft |
| 1817320 | Touchscreen Monitor | 21.5" touchscreen monitor |
| 1817319 | Mini PC | Mini PC |

Recommended List of Microscopes and the Base Stand Types

| Stereomicroscope Provisions | |
|-----------------------------|----------------------------|
| Microscope Types | Base stand |
| Esco Stereomicroscope MS-1 | Standard 1320375 |
| Nikon SMZ series | C-PS / C-PS160 / C-DSS |
| Olympus SZX series | SZX-ST / SZX2-ST/SZ2-ST |
| Carl Zeiss | Stand N / V8 with T-Pillar |
| Leica | S8AP0 |
| Optica | SZM-2/SZM-D |

ESCO LIFESCIENCES GROUP



-  Global Offices
-  Distributors
-  Factories
-  R&D Centers
-  Regional Distribution Centers

Esco Medical Products:

MIRI® Multiroom Incubator
 MIRI® Humidity Multiroom Incubator
 MIRI® II-12 Multiroom Incubator
 Mini MIRI® Dry Multiroom Incubator
 Mini MIRI® Humidity Multiroom Incubator

MIRI® TL6 Time-Lapse Incubator
 MIRI® TL12 Time-Lapse Incubator
 Multi-Zone ART Workstation
 MIRI® Laminar Flow Cabinet
 MIRI® Evidence RFID Witnessing & Traceability System

CelCulture® CO₂ Incubator
 MIRI® GA (Gas and Temperature Validation Unit)
 MIRI® AVT
 CultureCoin®

Infertility is a problem that has a significant social, psychological, and economic impact on afflicted individuals and couples. It is a global concern that knows no race or creed. It has been estimated that 1 in 6 couples struggle with infertility at least once in their lifetime.

Esco Medical is one of the divisions of the Esco Lifesciences Group. We provide innovative technological solutions for fertility clinics and laboratories. We aim to become the leading manufacturer of high-quality equipment such as long-term embryo incubators, ART workstations, anti-vibration tables, and time-lapse incubators.

Our products are designed with the Silent Embryo Hypothesis as a guiding principle. The Silent Embryo Hypothesis states that the less disturbed an embryo can remain, the better its developmental potential will be. Most of our products are designed in Denmark and made in the EU. Our primary focus is to increase pregnancy success rates and patient satisfaction.



Esco Medical, Aps

Kringelled 10, 8250 Egaa, Denmark
 medical@escolifesciences.com
www.esco-medical.com

Esco Micro, Pte. Ltd. (Headquarters)

19 Changi South Street 1, Singapore 486 779
 Tel +65 6542 0833

Esco Global Offices: Bangladesh | China | Denmark | Germany | Hong Kong | Indonesia | Lithuania | Malaysia | Myanmar | Philippines | Russia | Singapore | South Africa | South Korea | Taiwan | Thailand | UAE | UK | USA | Vietnam

